Steam Inhaler Inhalador de Vapor Inhalateur Vapeur

**USE AND CARE MANUAL
MANUAL DE USO Y CUIDADO
MANUEL D'UTILISATION ET D'ENTRETIEN**

This steam inhaler enables the user to breathe in warm water vapor for better breathing and a more comfortable sleep. If you have any questions about the operation of your Vicks Steam Inhaler, call our toll-free Consumer Relations line at 1-800-VAPOR-1-2 or e-mail us at Consumerrelations@kaz.com.

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IMPORTANT! READ AND SAVE THESE INSTRUCTIONS

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Modelo V1300N

Modèle V1300-CAN

Congratulations on your purchase of the **Vicks Steam Inhaler**. Your new Inhaler provides all the benefits of vapor therapy at home, at the office or when traveling.

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Directions for Use

Rinse unit out thoroughly before first use.

1. Remove top of unit from base by twisting in opposite directions and lifting off top.
2. Fill base to water line with hot (not boiling) tap water. NOTE: If you overfill, the unit will leak.
3. Open Vicks VapoPad[®] packet and add to the water. As an alternative, one teaspoonful of Vicks VapoSteam[®] or Kaz Inhalant can also be used. Replace the top.
4. Breathe in the soothing Vicks vapors.

CAUTION: Product contains hot water. To avoid hot water spilling after filling, be sure to properly secure top housing by twisting completely so it snaps back into place.

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Care and Cleaning

After each use, thoroughly wash and dry your inhaler. The unit is top shelf dishwasher safe.

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Use with Vicks VapoPads® or Vicks VapoSteam®

Your inhaler was designed to work with Vicks VapoPads®, Vicks VapoSteam® or Kaz Inhalant. The menthol and eucalyptus vapors from these provide soothing comfort.

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Recommended Usage for the Vicks Menthol VapoPads (VSP-19/VSP-19-CAN):

- Not recommended for infants under 10 lbs (4.5 kgs).
 - For children between 10 – 22 lbs (4.5-10 kgs) do not use more than 2 pads in a 24 hour period.
- VapoPad Ingredients: Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others.

⚠ WARNING: Do not rub face or eyes after touching the VapoPad. Oils in the pad could act as an irritant to eyes.

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One Year Limited Warranty

Your Vicks Steam Inhaler is warranted against defective materials or workmanship for one year from date of purchase. Any defective part will be replaced at no charge if it has not been tampered with and the appliance has been used according to our printed directions. This warranty gives you specific legal rights, as well as other rights which may vary from state to state and from province to province.

For questions or comments

Call us toll-free at: 1-800-VAPOR-1-2 (1-800-827-6712)

E-mail: consumerrelations@kaz.com

Or visit our website at: www.vickshumidifiers.com

Please be sure to specify the model number V1300

NOTE: IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE PRODUCT TO THE ORIGINAL PLACE OF PURCHASE.

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Helen of Troy

CREATIVE DEPARTMENT ARTWORK SPECIFICATIONS

Brand: Vicks
Category: Inhaler
Model: **V1300 series**
Artwork Part #: A001339R1
Die Line Part #: N/A
Description: OWNER'S MANUAL, KAZ Artwork,
V1300 Series
Subject: OM
Region: US
Size: MM: W50 x H25

Scale: 1/1
Material: 70g wood

Page count: 4
Revision: 2

Date: **17MAR17**

Release Date: 17MAR17

Rerelease Date:

Colors:

**Quality Requirement of Artwork and Quality
Clarification Process of Artwork Printing:
Meet Eng-QS-06&02**

Kaz USA, Inc., a Helen of Troy Company
Creative Services
Marlborough, MA, USA 01752
+1 508 490 7000

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© 2017 All rights reserved • Todos los Derechos Reservados • Tous droits réservés.
VICKS and other associated trademarks are owned by The Procter & Gamble Company • VICKS y otras marcas registradas asociadas son propiedad de The Procter & Gamble Company • VICKS et les marques de commerce connexes appartiennent à la société Procter & Gamble.